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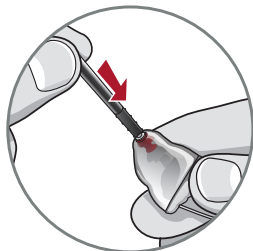
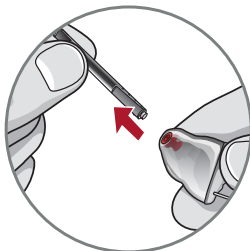
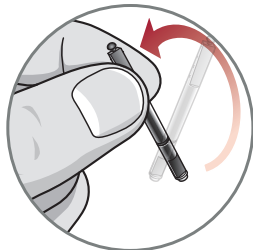
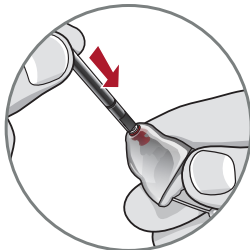
۴

Troubleshooting Guide

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
Not Loud Enough	Blocked microphone or receiver	Clean or replace wax guard as needed
	Hearing change	Contact your hearing professional
	Debris buildup	Clean both microphone and receiver with brush
Inconsistent Performance	Blocked microphone or receiver	Clean or replace wax guard as needed
Unclear, Distorted Performance	Blocked microphone or receiver	Clean or replace wax guard as needed
	Blocked vent	Clean vent
	Defective hearing aid	Contact your hearing professional
Dead	Charge required	Place in your charger until the LEDs stop blinking and become solid
	Blocked microphone or receiver	Clean or replace wax guard as needed

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Charger Troubleshooting Guide

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
Hearing aids whistling in the charger	Incorrect orientation	Reposition your hearing aids faceplate facing down on the charging post. Confirm that the charging LED begins to glow.
	Dead battery	Your charger's on-board battery has died. Plug your charger in. If you do not have the cord with you, push and hold the hearing aid's push button for 3 seconds to turn it off. This will conserve the charge in your hearing aid.

Charger Troubleshooting Guide

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
No LED indicator when hearing aids are inserted onto the charging posts	Dead battery	Connect the microUSB to your charger and plug it into the wall. The on-board battery LEDs will turn on for a few seconds to indicate connection to the power source. If they do not, contact your hearing professional.
	Incorrect orientation	Reposition your hearing aids on the post with the faceplate facing down on the charging post in charger reservoir. There is not a right and left post, your hearing aid will charge in either post.
Red blinking LED by charging posts	Fault occurred	Remove the hearing aids from the charging post, wait until the LED turns off, reinsert. If the red blinking LED persists, contact your hearing professional.
While charging cordless, no LEDs are illuminated	Power save mode	To refresh LEDs, remove a hearing aid from the charging post for 3 seconds and then reposition the hearing aids on the charging post. Both the charging LEDs and the on-board battery LEDs will illuminate

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Use on Aircrafts*

The optional wireless capabilities that may be featured in your hearing aids can be used on an aircraft as hearing aids are exempt from the rules applied to other personal electronic devices on an aircraft.

International Use*

The optional wireless capabilities that may be featured in your hearing aids are approved to operate at a radio frequency specific to your country or region and might not be approved for use outside your country or region. Be aware that operation during international travel may cause interference to other electronic devices, or other electronic devices may cause interference to your hearing aids.

We are required by regulations to provide the following warnings:

⚠ WARNING: Use of wireless hearing aids directly next to other electronic equipment should be avoided because it could result in improper performance. If such use is necessary, note as to whether your hearing aids and the other equipment are operating normally.

⚠ WARNING: Use of accessories, components or replacement parts other than those provided by the manufacturer of your hearing aids could result in increased electromagnetic emissions and decreased electromagnetic immunity and could result in degradation of performance.

⚠ WARNING: If Portable Radio Frequency communications equipment is used closer than 30 cm (12 inches) from your hearing aid, degradation of the performance of your hearing aid could result. If this occurs, move away from the communications equipment.

DO NOT OPEN HEARING AID OR CHARGER, NO USER-SERVICEABLE PARTS INSIDE

The Starkey Hearing Technologies Charger has a rating of IP 5X per IEC 60529. This means that the Starkey Hearing Technologies Charger is protected from dust.

The service life of the Starkey Hearing Technologies Charger is 3 years.

*Applies to wireless hearing aids only.

Safety Information

INTENDED USE: An air conduction hearing aid is a wearable sound-amplifying device intended to compensate for impaired hearing. Hearing aids are available in multiple gain/output levels appropriate to treat hearing losses ranging from mild to profound.

Your hearing aids are designed to comply with the most stringent Standards of International Electromagnetic Compatibility. However, it is still possible that you may experience interference caused by power line disturbances, airport metal detectors, electromagnetic fields from other medical devices, radio signals and electrostatic discharges.

If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your hearing aids might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance.

Your hearing aids should not be worn during an MRI procedure or in a hyperbaric chamber.

Your hearing aids are classified as a Type B applied part under the IEC 60601-1 medical device standard.

Your hearing aids are not formally certified to operate in explosive atmospheres that may be found in coal mines or certain chemical factories.

 Your hearing aids and charger should be stored within the temperature and humidity ranges of -10°C (14°F) to +45°C (113°F) and 10%-95% rH.

The charging temperature range is between 0°C (32°F) and 40°C (104°F).

Your hearing aids are designed to operate beyond the range of temperatures comfortable to you, from very cold up to 40°C (104°F).

Any serious incident that has occurred in relation to your Starkey device should be reported to your local Starkey Hearing Technologies representative and the Competent Authority of the Member State in which you are established. A serious incident is defined as any malfunction, deterioration in the characteristics and/or performance of the device, or inadequacy in the device Operations Manual/ labeling which could lead to the death or serious deterioration in the state of health of the user, OR could do so upon recurrence.

⚠ CAUTION: If the charger is hot, do not touch it until cool. If the product is not working, do not disassemble. Due to a shock hazard, please send in for repair. Keep out of reach from children. Do not ingest anything in the package, including desiccant, cleaning tool, etc. Any cords and AC adapters must be approved or listed by a Nationally Recognized Testing Laboratory.

If you have reservations about your ability to adapt to amplification, you should inquire about the availability of a trial-rental or purchase-option program. Many hearing aid dispensers now offer programs that permit you to wear a hearing aid for a period of time for a nominal fee after which you may decide if you want to purchase the hearing aid.

Federal law restricts the sale of hearing aids to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

A hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Use of a hearing aid is only part of hearing habilitation and may need to be supplemented by auditory training and instruction in lip reading. In most cases infrequent use of a hearing aid does not permit a user to attain full benefit from it. Special care should be exercised in selecting and fitting a hearing aid whose maximum sound pressure level exceeds 132 decibels because there may be risk in impairing the remaining hearing of the hearing aid user.

Some hearing aid users have reported a buzzing sound in their hearing aid when they are using mobile phones, indicating that the mobile phone and hearing aid may not be compatible. According to the ANSI C63.19 standard (ANSI C63.19-2007 American National Standard Methods of Measurement of Compatibility Between Wireless Communications Devices and Hearing Aids), the compatibility of a particular hearing aid and mobile phone can be predicted by adding the rating for the hearing aid immunity to the rating for the mobile phone emissions. For example, the sum of a hearing aid rating of 2 (M2/T2) and a telephone rating of 3 (M3/T3) would result in a combined rating that equals at least 5 would provide "normal use"; a combined rating of 6 or greater would indicate "excellent performance". See the Product Card or Quick Start Guide included with your hearing aid for the exact M/T rating of your hearing aid.

CHILDREN WITH HEARING LOSS

In addition to seeing a physician for a medical evaluation, a child with a hearing loss should be directed to an audiologist for evaluation and rehabilitation since hearing loss may cause problems in language development and the educational and social growth of a child. An audiologist is qualified by training and experience to assist in the evaluation and rehabilitation of a child with a hearing loss.

Required Hearing Aid Information

The following additional information is provided in compliance with U.S. Food and Drug Administration (FDA) regulations:

WARNING TO HEARING AID DISPENSERS

A hearing aid dispenser should advise a prospective hearing aid user to consult promptly with a licensed physician (preferably an ear specialist) before dispensing a hearing aid if the hearing aid dispenser determines through inquiry, actual observation or review of any other available information concerning the prospective user that the prospective user has any of the following conditions:

- i. Visible congenital or traumatic deformity of the ear.
- ii. History of active drainage from the ear within the previous 90 days.
- iii. History of sudden or rapidly progressive hearing loss within the previous 90 days.
- iv. Acute or chronic dizziness.
- v. Unilateral hearing loss of sudden or recent onset within the previous 90 days.
- vi. Audiometric air-bone gap equal to or greater than 15 decibels at 500 Hertz (Hz), 1,000 Hz and 2,000 Hz.
- vii. Visible evidence of significant cerumen accumulation or a foreign body in the ear canal.
- viii. Pain or discomfort in the ear.

IMPORTANT NOTICE FOR PROSPECTIVE HEARING AID USERS

Good health practice requires that a person with a hearing loss have a medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear) before purchasing a hearing aid. Licensed physicians who specialize in diseases of the ear are often referred to as otolaryngologists, otologists or otorhinolaryngologists. The purpose of the medical evaluation is to assure that all medically treatable conditions that may affect hearing are identified and treated before the hearing aid is purchased.

Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for a hearing aid.

The physician will refer you to an audiologist or hearing aid dispenser, as appropriate, for a hearing aid evaluation.

The audiologist or hearing aid dispenser will conduct a hearing aid evaluation to assess your ability to hear with and without a hearing aid. The hearing aid evaluation will enable the audiologist or dispenser to select and fit a hearing aid

⚠ CAUTION: If set to the maximum output level and worn for periods of time exceeding the recommendations below, the patient's exposure to sound energy has the potential to exceed noise exposure limits. This device is intended for use for a maximum of sixteen (16) hours a day when set at the maximum output level.

For the Patient

A tinnitus therapy device is an electronic device intended to generate noise of sufficient intensity and bandwidth to treat ringing in the ears. It can also be used as an aid in hearing external sounds and speech.

Multiflex Tinnitus Technology is a tool to generate sounds. It is recommended that this tool be used with appropriate counseling and/or in a tinnitus management program to relieve patients suffering from tinnitus.

TINNITUS THERAPY CONCEPTS AND BENEFITS

Multiflex Tinnitus Technology can be used as a part of a tinnitus treatment program.

Multiflex Tinnitus Technology plays a white noise through the hearing aid.

Multiflex Tinnitus Technology is programmed according to your hearing loss and preference, and your hearing professional can adjust the settings of Multiflex Tinnitus Technology to meet your needs.

Multiflex Tinnitus Technology may provide temporary relief of your tinnitus.

PRESCRIPTION USE ONLY

⚠ CAUTION: Federal law restricts this device to sale by or on the order of a doctor, audiologist or other hearing care practitioner licensed to dispense hearing aids in your state.

The use of any sound generating tinnitus therapy device should be only on the advice and in consultation with your audiologist or hearing care practitioner. Your hearing professional will properly diagnose and fit the device to your personal needs and requirements. This should include its use in a prescribed tinnitus treatment program.

Your hearing professional will also be able to offer the appropriate follow-up care. It is important that you follow your hearing professional's advice and direction regarding such care.

⚠ WARNING: There are some potential concerns associated with the use of any sound generating tinnitus therapy device. Among them are the potential for worsening of tinnitus, a possible change in hearing thresholds and possible skin irritation at the point of contact with the device.

Multiflex Tinnitus Technology has been designed to minimize these concerns. However, should you experience or notice any of the above conditions or any dizziness, nausea, headaches or heart palpitations, you should immediately

For Hearing Professionals

INDICATIONS FOR USE

The Multiflex Tinnitus Technology is a tool to generate sounds to be used in a Tinnitus Management Program to relieve patients suffering from tinnitus. The target population is primarily the adult population over 18 years of age.

The Multiflex Tinnitus Technology is targeted for healthcare professionals, which are treating patients suffering from tinnitus, as well as conventional hearing disorders. The fitting of the Multiflex Tinnitus Technology must be done by a hearing professional participating in a Tinnitus Management Program.

DEVICE DESCRIPTION

Multiflex Tinnitus Technology is a software function that generates sound which is programmed into a hearing aid. The hearing aid may be used in one of three modes of operation: as a hearing aid, as a tinnitus treatment device or as a hearing aid and tinnitus treatment device.

When enabled, the Multiflex Tinnitus Technology generates the sound and allows the patient's hearing professional to design and program appropriate settings for an individually prescribed sound treatment plan. The treatment plan should be used in a tinnitus management program for relief of tinnitus.

Multiflex Tinnitus Technology generates a broadband white noise signal that varies in frequency and amplitude. These characteristics are adjustable by the hearing professional and are specific to the prescribed therapy designed by the hearing professional for the patient's needs and comfort.

The patient may have some control of the level or volume of the signal and the patient should discuss this adjustment as well as his or her comfort level and sound of the signal with their hearing professional.

⚠ WARNING TO HEARING CARE PRACTITIONER

A hearing care practitioner should advise a prospective sound generator user to consult promptly with a licensed physician (preferably an ear specialist) before using a sound generator if the hearing care practitioner determines through inquiry, actual observation or review or any other available information concerning the prospective user that the prospective user has any of the following conditions:

- Visible congenital or traumatic deformity of the ear.
- History of active drainage from the ear within the previous 90 days.
- History of sudden or rapidly progressive hearing loss within the previous 90 days.
- Acute or chronic dizziness.
- Unilateral hearing loss of sudden or recent onset within the previous 90 days.

The Starkey Hearing Technologies Charger has been tested to, and has passed the following emissions and immunity tests:

- IEC 60601-1-2 radiated and conducted emissions requirements for a Group 1 Class B device as stated in CISPR 11.
- Harmonic distortion and voltage fluctuations affecting the power input source as stated in Table 2 of IEC 60601-1-2.
- RF radiated immunity at a field level of 10 V/m between 80 MHz and 2.7 GHz as well as higher field levels from communications devices as stated in Table 9 of IEC 60601-1-2.
- Immunity to power frequency magnetic fields at a field level of 30 A/m.
- Immunity to ESD levels of +/- 8 kV conducted discharge and +/- 15 kV air discharge.
- Immunity to Electrical fast transients on the power input at a level of +/- 2 kV at a 100 Hz repetition rate.
- Immunity to Surges on the power input of +/- 1 kV line to line.
- Immunity to conducted disturbances induced by RF fields on the power input as stated in Table 6 of IEC 60601-1-2.
- Immunity to voltage dips and interruptions on the power input as stated in Table 6 of IEC 60601-1-2.

REGULATORY NOTICES

FCC ID: EOA-24LIVIOITEL (left), EOA-24LIVIOITER (right)

IC: 6903A-24LIVIOITEL (left), 6903A-24LIVIOITER (right)

FCC NOTICE

This device complies with part 15 of the FCC rules and with ISSED Canada's license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device.

Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

Hereby, Starkey Hearing Technologies declares that the Custom hearing aid is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU A copy of the Declaration of Conformity can be obtained from the addresses on the following page or from docs.starkeyhearingtechnologies.com

As with any device, misuse of the tinnitus therapy device could present some potentially harmful effects. Care should be taken to prevent the unauthorized use and to keep the device out of the reach of children and pets.

⚠ CAUTION: If set to the maximum output level and worn for periods of time exceeding the following recommendations, your exposure to sound energy has the potential to exceed noise exposure limits. You should not use your hearing device for more than sixteen (16) hours a day if your device is set at the maximum output level, nor should you use your device if your hearing professional has set the device at levels that exceed your comfort level.

Important Notice for Prospective Sound Generator Users

Good health practice requires that a person with tinnitus have a medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear) before using a sound generator. Licensed physicians who specialize in diseases of the ear are often referred to as otolaryngologists, otologists or otorhinolaryngologists.

The purpose of a medical evaluation is to assure that all medically treatable conditions that may affect tinnitus are identified and treated before the sound generator instrument is used.

TINNITUS TECHNICAL DATA

Multiflex Tinnitus Technology Maximum Output = 87 dB SPL (typical) when measured in a 2cc coupler per ANSI S3.22 or IEC 60118-7.

WIRELESS TECHNICAL DESCRIPTION

Your hearing aids contain a radio transceiver utilizing Bluetooth Low Energy wireless technology operating in the 2.4-2.4835 GHz frequency band with a maximum effective radiated power of -4 dBm using GFSK transmission modulation. The receiver section of the radio has a bandwidth of 1.5 MHz.

This hearing aid model has been tested to, and has passed, the following emissions and immunity tests:

- IEC 60601-1-2 radiated emissions requirements for a Group 1 Class B device as stated in CISPR 11
- RF radiated immunity at a field level of 10 V/m between 80 MHz and 2.7 GHz as well as higher field levels from communications devices as stated in Table 9 of IEC 60601-1-2
- Immunity to power frequency magnetic fields at a field level of 30 A/m
- Immunity to ESD levels of +/- 8 kV conducted discharge and +/- 15 kV air discharge

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Class II Device

Waste from electronic equipment must be handled according to local regulations

Consult Operations Manual

Keep dry

INSTRUCTIONS FOR DISPOSAL OF OLD ELECTRONICS

Starkey Hearing Technologies encourages, the EU requires, and your local community laws may require, that your hearing aids and charger be disposed of via your local electronics recycling/disposal process.

The instructions below are provided for the benefit of disposal/recycling personnel. Please include this manual when disposing of your hearing aids and/or charger.

FOR DISPOSAL/RECYCLING PERSONNEL ONLY

These products contain Lithium Ion Polymer batteries. For instructions on removing the battery from the hearing aids, please visit docs.starkeyhearingtechnologies.com. To remove the battery from the Starkey Hearing Technologies Charger:

- Remove the four skid pads on the bottom of the charger base to access fasteners.
- Remove fasteners using a Phillips head screwdriver.
- Disassemble the charger base from the bezel to expose the battery cell.
- Cut the THREE battery wire leads ONE AT A TIME close to the battery cell to avoid shorting.
- Pry battery from base using a wide flat blade making sure not to puncture the battery cell.

Starkey, Starkey Hearing Technologies, Thrive and Just Talk are trademarks of Starkey Laboratories, Inc.

Apple and iPhone are trademarks of Apple Inc. registered in the U.S. and other countries.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Starkey is under license.

iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license. Android is a trademark of Google LLC.

©2021 Starkey. All Rights Reserved. 86374-007 3/21 BKLT3088-03-EE-XX

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Starkey Laboratories (Germany) GmbH